




# MORE PATIENTS CAN DO IT WITHOUT LANCETS<sup>1</sup>

Moving your patients from the FreeStyle Libre system to the FreeStyle Libre 2 system?

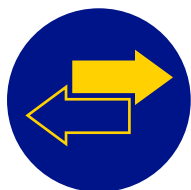

We're pleased to introduce you to the second generation of our sensor-based technology for people living with diabetes. The FreeStyle Libre 2 system comes with the same great features of the FreeStyle Libre system and more.

  
**FreeStyle**  
**Libre 2**  
FLASH GLUCOSE MONITORING SYSTEM

  
*life. to the fullest.®*  
**Abbott**

Images are for illustrative purposes only. Not actual patient or data.  
1. Scanning the sensor does not require lancets.

# Moving your patients to the FreeStyle Libre 2 system is easy. Follow the guidance below.


**CHANGE THE PATIENT'S PRESCRIPTION TO  
FREESTYLE LIBRE 2 SENSORS**



**WHICH DEVICE WOULD YOUR PATIENT LIKE TO RECEIVE  
ALARMS ON?¹**



**PHONE**

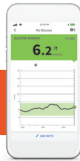

## **THEY'RE READY TO GO!**

Using the FreeStyle LibreLink app, patients can start scanning FreeStyle Libre 2 sensors straight away<sup>2</sup>

Patient to check that their FreeStyle LibreLink app version is 2.5 or above. See compatibility guide for details.



**READER**



## **REQUEST A READER**

Patient goes to the Reader Replacement Portal and completes the online request form

[FreeStyleLibre.co.uk/replacement](https://www.freestylelibre.co.uk/replacement)



## **REQUEST VALIDATION**

- Patient enters their current FreeStyle Libre reader serial number
- No requirement to send the current reader back



## **DELIVERED DIRECTLY TO THE PATIENT**

A replacement reader is delivered directly to the FreeStyle Libre user



Images are for illustrative purposes only. Not actual patient data.

**1.** Patients choose which device they want to receive alarms: FreeStyle Libre 2 reader or FreeStyle LibreLink app. They must start their FreeStyle Libre 2 sensor with that selected device. Once the patient scans their FreeStyle Libre 2 sensor with that device, they can receive alarms only on that device. **2.** The FreeStyle LibreLink app is only compatible with certain mobile devices and operating systems. Please check the website for more information about device compatibility before using the app. Use of FreeStyle LibreLink requires registration with LibreView.

## EDUCATION AND SUPPORT FROM ABBOTT – SUPPORTING USERS THROUGH THE MOVE



FreeStyle Academy



Webinars



Website information



New starter guide



Training presentations for peer-to-peer  
and patients

## Prescribing information.



### THE FREESTYLE LIBRE 2 SENSOR

Ask your local Abbott representative for a  
Prescription Change Request Letter Template  
to send to GPs.

PIP Code: 416-3416

Discover more support and guidance  
for you and your patients.

Visit us at: **[www.FreeStyleDiabetes.co.uk](http://www.FreeStyleDiabetes.co.uk)**

# What is the difference between the FreeStyle Libre system and the FreeStyle Libre 2 system?

Just like the FreeStyle Libre system, the FreeStyle Libre 2 system is discreet, convenient and easy to use<sup>1</sup>, now with added benefits:



## No finger pricks<sup>2</sup>

There's no need to prick your finger when using the FreeStyle Libre 2 system - even when glucose is low, falling or rapidly changing.



## Excellent accuracy<sup>1,3</sup>

Outstanding accuracy in the low glucose range when it matters most



## Optional alarms

Peace of mind when it's needed most, with three optimal real-time alarms for those who need them.

- Low glucose alarm
- High glucose alarm
- Signal loss alarm

  
FreeStyle  
Libre 2  
FLASH GLUCOSE MONITORING SYSTEM

  
*life. to the fullest.®*  
Abbott

Images are for illustrative purposes only.

**1.** Data on file, Abbott Diabetes Care, Inc. **2.** Finger pricks are required if your glucose readings and alarms do not match symptoms or expectations. **3.** Alva S, et al. Accuracy of a 14-Day Factory-Calibrated Continuous Glucose Monitoring System With Advanced Algorithm in Pediatric and Adult Population With Diabetes. Journal of Diabetes Science and Technology. September 2020. doi:10.1177/1932296820958754.

© 2020 Abbott. FreeStyle, Libre, and related brand marks are marks of Abbott. ADC-32038 v1.0 12/20.